


## Enuresis Monitor

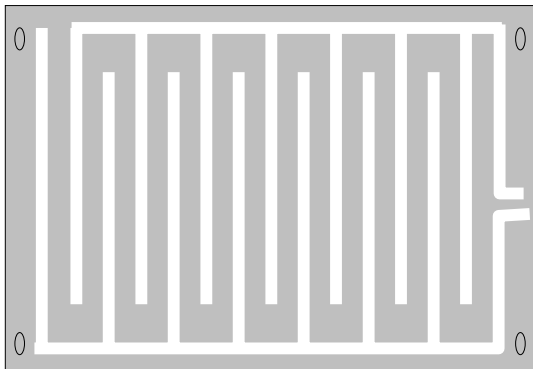

### STX-ENU

Designed to detect Bed Wetting. The Bed-mat is made of foil embossed plastic, which fits between the bottom sheet and the mattress.

Once the mat has become moist, an alarm will be triggered. New replacement Bed-Mats are available.

## Instructions for Use

- Place the Bed-Mat over the mattress in the area that is normally wetted and tie down.
- Do not use waterproof or water resistant sheeting above mat!
- Cover the Bed-Mat with a clean dry sheet and make up bed as usual.
- When the alarm is activated, try and stop any more urine from being released and use the lavatory to completely empty the bladder.
- The Bed-Mat must be wiped clean with detergent and dried, then replaced in a freshly made bed, as before for reuse.