

WristCare Automatic Personal Alarm



The Wristcare transmitter is worn like a watch. It is designed to monitor the user's well being.

Wristcare contains sensors which measure movement, sweat level, skin temperature and pulse. When the unit detects signs of a significant change from the user's norm it will send an alarm, which activates the pager being carried by the carer (or other alert device)

The user can generate an emergency call by pressing the grey button on the Wristcare.

Wristcare is splash resistant but must be removed when bathing or showering.

The Wristcare must be worn in bed at night. Do not remove the unit when leaving the house.

The Wristcare will also transmit an alarm when it is more than 20 metres from the case or if it has been removed from the user's wrist for more than 30 minutes.

Alarm Options

Local Alarm

A small unit is located in the sleepover room which responds to the device being activated.



Numeric and Alphanumeric Paging Systems

A pager is carried by the carer and will beep when it receives a signal from the monitor. This option is suited to larger nursing homes or hospitals as the range is much greater than the mains powered radio receivers. Another advantage of this option is that the carer can prioritise their response.



Nursecall System

The bed monitor can be supplied with a link that allows it to interface with an existing nursecall system in hospitals or nursing/residential homes.

