

# Product Instructions

## GEO28V3MC

### Rechargeable 4 Line Alphanumeric Pager (AAA)

Turn the pager ON by pressing and holding '4' for two seconds and release, it beeps, press '4' again. The current date and time is displayed (this does not require to be set for the pager to function correctly).



The pager is carried by the carer and will bleep or vibrate when it receives a signal from the transmitter, pressing '4' ACCEPTS the call and silences the pager.

The pager has a working range of approx. 300m from the paging transmitter.

For the SPTX range of equipment the range is approx. 30 – 50m.

Old messages of activations, which are stored in the pager's memory, can be viewed by pressing '4' continuously until all of the old messages have been read.

To delete old messages, view all messages, from the Standby Screen, press '4' once, press '1' once, press '3' twice and '4' once. 'DELETE?' is displayed, press '3' twice once (flashing 'tick'), press '4' once. If files successfully deleted, a victory beep is heard.



To turn pager off, (when in standby mode, showing time and date) press the '1' button once, then '2' once, followed by '4'. A victory beep is heard and the pager screen goes blank.

The GEO28V3MC has a power saving device which turns the LCD screen off, the pager comes into life when it receives a message or when any of the buttons are pressed.

The unit is supplied with a rechargeable AAA battery and a Micro – Standard USB cable and mains charger. Refer to pager manual for further information.