

PIR2

Portable Movement Detector

- A passive monitors which plugs into the jack socket on the room's nurse call point.
- Typically the unit is used to detect a person moving within a room or down a corridor.
- The lens is masked so that a narrow beam is projected from the middle of the lens making the monitoring of movement more precise.



1. Remove the cover screw

2. Remove the PCB screw at the top of the PIR

3. Remove the PCB and lay it to the side without disconnecting any wires

4. Drill a hole in the back box and screw the unit to the wall socket and reset the call point

5. Reassemble the unit & insert the plug into the call point

