


Enuresis

Incontinence Mat

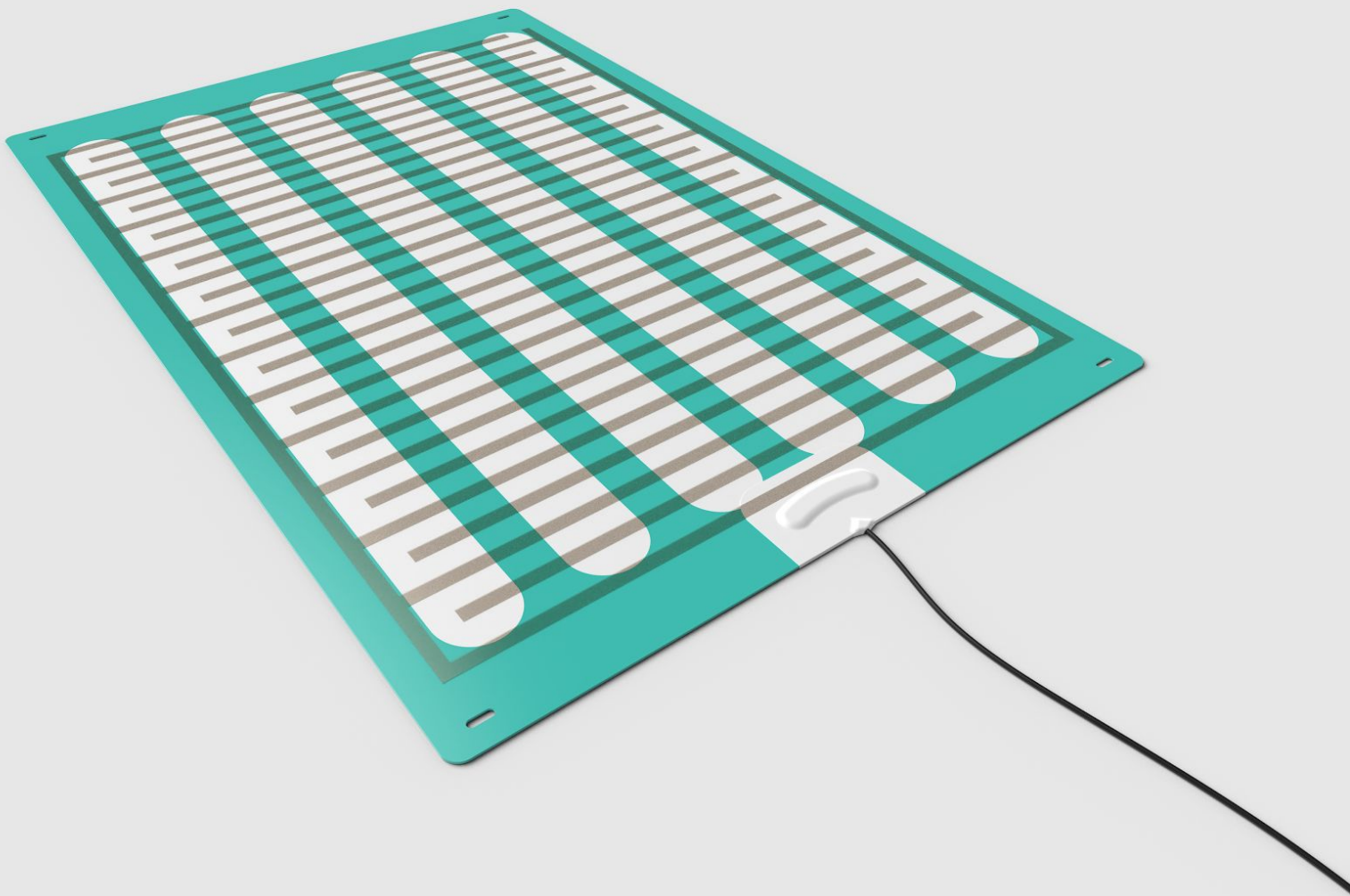

Placed on a bed, between the mattress and sheet, this mat provides immediate warning on detection of moisture, allowing effective action to be taken. The mat reduces the need for intrusive and undignified physical checks, promoting dignity and independence.

These durable and wipe-clean mats can be re-used for cost effective and hygienic implementation.

Specifications

Compatibility	Cair Detect, and most 3rd party devices
Dimensions	L760 x W510
Length of Cable	1900mm
Type Of Plug	RJ12
Warranty	12 months

Information

The Enuresis mat can be used with the Cair Detect and most 3rd party devices to alert when moisture is detected. It should be placed flat on top of the mattress with the green and white side facing upwards, above any waterproof sheet, and positioned around the hip area. If the sensor is too sensitive, place another sheet on top of the mat. Ensure the cable is tucked safely under sheets or below the mattress to avoid it becoming a trip risk.

To test, simply apply a damp cloth to the green and white side for a few seconds until the connected device alerts. Periodically test the sensor to ensure it is functioning properly.

The mat should be cleaned regularly. To clean, use disinfectant wipes or apply some antibacterial solution to a damp cloth, take care to avoid damaging the area where the cable enters the mat. Allow to dry naturally or wipe with a clean, dry towel. Do not immerse the mat in any solution.

When the mat is in storage, keep flat or loosely roll the mat. Do not bend the mat.

