



Smart Technology, Made to Care

Sleep

Bed Occupancy Pad



Cair Sleep is a bed occupancy pad. When used in conjunction with an occupancy sensor like the Cair Detect, it can alert immediately when a user leaves the bed, or after a period of time if they don't return.

The pad, constructed using extra soft vinyl so it's less likely to be noticed by those using it, should be placed above the mattress.

Specifications

Table with 2 columns: Specification Name, Value. Rows include Compatibility (Cair Detect, and most 3rd party devices), Dimensions (L750 x W250), Length of Cable (2300mm), Type Of Plug (RJ12), and Warranty (12 months).

Information

The Sleep Bed Occupancy Pad can be used with the Cair Detect and most 3rd party devices. It should be placed above the mattress and below the sheet, and positioned around the hip area. Ensure the cable is tucked safely below the sheets or mattress to avoid it becoming a trip risk.

When testing, follow the instructions for the connected device. Periodically test the pad to ensure it is functioning properly.

The pad should be cleaned regularly. To clean, use disinfectant wipes or apply some antibacterial solution to a damp cloth and wipe the sensor clean, ensuring moisture doesn't enter the area where the cable enters the pad. Allow to dry naturally or wipe with a clean, dry towel. Do not immerse the pad in any solution.

The pad should be replaced after one year of use. When in storage, keep the pad flat. Do not bend the pad.

